

LIFEPAK® 500 automated external defibrillator OPERATOR'S CHECKLIST



Unit Serial No.: _____ Location: _____

Instruction	Recommended Corrective Action	Date							
		Initials							

1. Examine the AED case, connector, and battery well for:

Insert a ✓ in the box after completing each instruction.

Foreign substances

Clean the device.

--	--	--	--	--	--	--	--	--	--

Damage or cracks

Contact authorized service personnel.

--	--	--	--	--	--	--	--	--	--

2. Examine the battery pins for bending or discoloration.

Discard and replace battery.

--	--	--	--	--	--	--	--	--	--

3. Check expiration date on batteries and QUIK-COMBO electrodes.

Replace if expired.

--	--	--	--	--	--	--	--	--	--

4. Examine the accessory cables for cracked, damaged, broken, or bent connectors.

Replace damaged or broken parts.

--	--	--	--	--	--	--	--	--	--

5. With the battery installed, press ON/OFF to turn on the AED and look for:

Self-test messages

If absent, contact authorized service personnel.

--	--	--	--	--	--	--	--	--	--

Momentary illumination of each LED and all LCD segments

If absent, contact authorized service personnel to repair or replace parts.

--	--	--	--	--	--	--	--	--	--

BATTERY LOW or REPLACE BATTERY SELF-TEST xx:xx message

Replace the battery immediately.

--	--	--	--	--	--	--	--	--	--

Service Indicator or CALL SERVICE message

Contact authorized service personnel.

--	--	--	--	--	--	--	--	--	--

If additional blank checklist copies are required, reproduce this sheet before you fill in the information.